



Protective Safety Goggles

As the professional manufacture, we would like to provide you high quality Fealter® Protective Safety Goggles. Protective Safety Goggles used in medical institutions for protection during inspection and treatment, blocking body fluids, blood splashes or splashes.

You can rest assured to buy customized Fealter® Protective Safety Goggles from us. In recently years, the factory has won the sincere love of customers by relying on its sincere reputation, professional and and enthusiastic service to convey the three excellent standard services of high quality materials, preferential prices and excellent service to our customers.

Protective Safety Goggles

【Product Name】 Protective Safety Goggles

【Product Specifications】 1. with Exhalation Valve 2.without Exhalation Valve

【Product Description】

Polycarbonate (PC) lens, anti-fogging;
soft PVC frame fits your face comfortably;
Single pair of eye mask PE bag.

【Product Use】 Used in medical institutions for protect during inspection and treatment, to block bodily fluids, blood splatter or splatters, sand and gravel.

【Instructions Use】 Wear on eyes.

【Term of Use】 See packaging, valid for three years.

Five Functions of Protective Safety Goggles

1. Anti-splash
2. Anti-infection
3. Dust-proof
4. Anti-impact
5. Anti-UV

【Instructions for Use】

1. Avoid contact with sharp objects during use to avoid scratches and affect the protection effect. Keep it properly after use.

2.Wear it directly on the eyes and adjust the elastic band to a suitable state.

【Storage Conditions】 Store in a room with relative humidity $\leq 80\%$, temperature $-10^{\circ}\text{C} \leq 40^{\circ}\text{C}$, and no corrosive gas and good ventilation. It shall not be directly exposed to the sun, and shall be placed away from the heat source, and shall not contact with paint, acid and alkali, etc. which are harmful to the product.

【Warning】

1.Storage under the specified conditions, 3years.

2.When cleaning the product, use a soft cloth moistened with alcohol or neutral detergent to wipe the surface dirt of the product and use or store it after the surface is naturally dry.

If necessary, it can be sterilized with redox gas. Can not be sterilized by high pressure and high temperature.

3. Use should be in accordance with the relevant regulations, after use should be properly kept.

4. If breakage and cracks are found, use is prohibited.

【Waste Disposal】 Please do not discard this product after use, and handle it properly to avoid environmental impact.

Product Description

You can rest assured to buy customized Fealter® Protective Safety Goggles from us. In recent years, the factory has won the sincere love of customers by relying on its sincere reputation, professional and enthusiastic service to convey the three excellent standard services of high quality materials, preferential prices and excellent service to our customers.

Protective Safety Goggles

【Product Name】 Protective Safety Goggles

【Product Specifications】 1. with Exhalation Valve 2. without Exhalation Valve

【Product Description】

Polycarbonate (PC) lens, anti-fogging ;

soft PVC frame fits your face comfortably;

Single pair of eye mask PE bag.

【Product Use】 Used in medical institutions for protection during inspection and treatment, to block bodily fluids, blood splatter or splatters, sand and gravel.

【Instructions Use】 Wear on eyes.

【Term of Use】 See packaging, valid for three years.

Five Functions of Protective Safety Goggles

1. Anti-splash
2. Anti-infection
3. Dust-proof
4. Anti-impact
5. Anti-UV

【Instructions for Use】

Pingyang Peace Personal Protective Equipment Co., Ltd.

Email: peaceppe@outlook.com

Tel: +86-574-87394752

1. Avoid contact with sharp objects during use to avoid scratches and affect the protection effect. Keep it properly after use.

2. Wear it directly on the eyes and adjust the elastic band to a suitable state.

【Storage Conditions】 Store in a room with relative humidity $\leq 80\%$, temperature $-10^{\circ}\text{C} \leq 40^{\circ}\text{C}$, and no corrosive gas and good ventilation. It shall not be directly exposed to the sun, and shall be placed away from the heat source, and shall not contact with paint, acid and alkali, etc. which are harmful to the product.

【Warning】

1. Storage under the specified conditions, 3 years.

2. When cleaning the product, use a soft cloth moistened with alcohol or neutral detergent to wipe the surface dirt of the product and use or store it after the surface is naturally dry. If necessary, it can be sterilized with redox gas. Can not be sterilized by high pressure and high temperature.

3. Use should be in accordance with the relevant regulations, after use should be properly kept.

4. If breakage and cracks are found, use is prohibited.

【Waste Disposal】 Please do not discard this product after use, and handle it properly to avoid environmental impact.

