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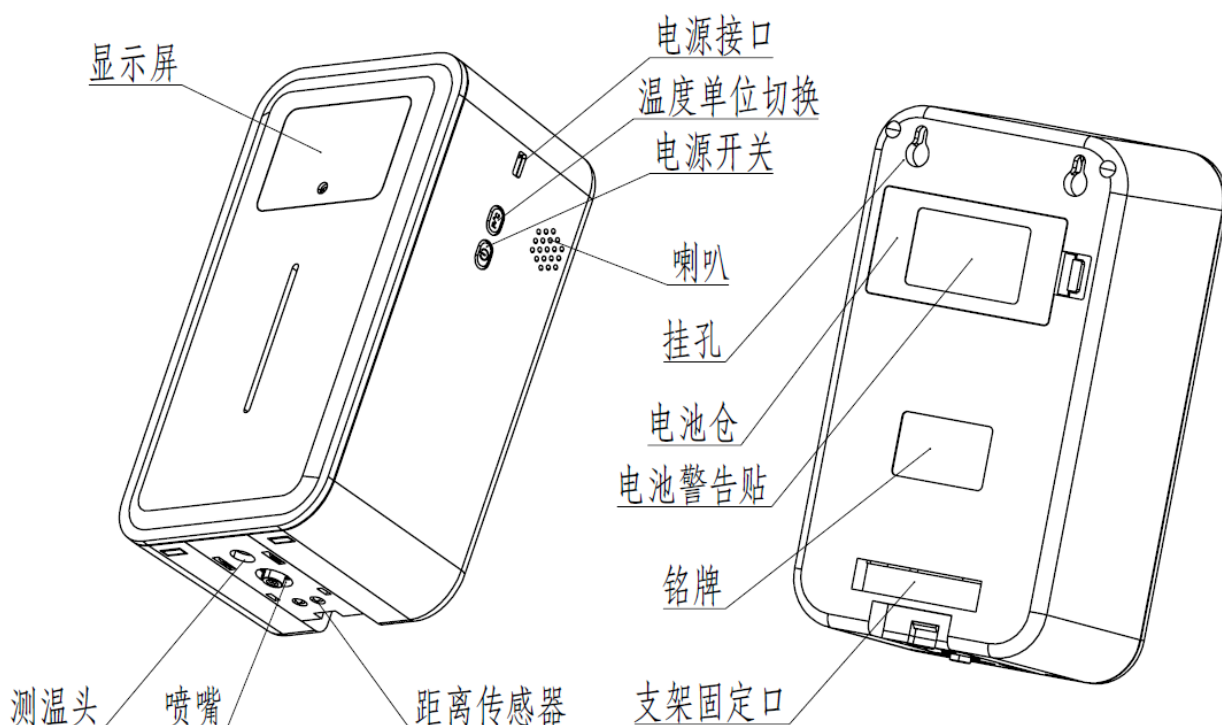
Alarm: abnormal alarming automatically, sound alarm with light flashing

3. 测量距离: 约 5cm 自动测量  
Measuring distance: ~5cm auto-measuring
4. 显示屏: 数字显示  
Display: LCD display with figure
5. 传输接口: USB type c 接口  
Transmission interface: USB Type C
6. 装液瓶容量: 450mL  
Liquid capacity: 450mL
7. 安装方式: 壁挂、支架固定  
Installment: wall-hanging, bracket fixing
8. 工作温度: 10°C ~40°C (推荐 15°C~35°C )  
Working Environment : 10°C~40°C (recommended 15°C~35°C)
9. 测量范围: 10~50°C  
Measuring range: 10~50°C
10. 响应时间: 500ms  
Response time: 500ms
11. 待机方式: 非工作时间 5S 待机, 白点动态循环  
Standby: 5S standby during non-working condition with dynamic cycle of flashing white dot
12. 输入电压: DC 5V 2A(可内置 2 节 18650 锂电池供电)  
Input: DC 5V 2A (or 2\*18650 li-ion battery)
13. 外型尺寸: 208X115X86mm  
Dimension: 208\*115\*86mm
14. 液体种类: 液态洗手液, 液态消毒水  
Liquid type: liquid soap, liquid disinfectant

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Standby: The white dot on the lower right corner of the display flashing in turns. (Fig.1)

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Normal temperature: Current temperature displayed, with light flashing green once. (Fig.2)

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Abnormal temperature: Current temperature displayed, with alarming and light flashing red. (Fig.3)

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Out of measuring range: "Hi" displayed means current measured temperature is much higher than the settings. (Fig.4)

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1. 测温洗手：将手伸进设备底部 (图 5)，向其靠近可听到嘀一声就可测出温度数据及自动喷出洗手液。  
Temperature measuring and hands washing: Put hands to the bottom until a "DI" sounds, the current temperature measured and displayed on the screen, hand sanitizer squeezed out. (Fig.5)

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Press button① once to set the temperature unit

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Press button② to switch unit between Celsius (Fig.8) and Fahrenheit (Fig.9)

按 2 下按键①：调工作模式 Press button① twice to set the working mode

按键②切换 人体模式，计数模式，表面模式 Press button② to switch the working mode. BO: Body mode, CO: Counting mode. SUR: Surface mode

按 3 下按键①：调报警值 Press button① 3 times to set the alarm temperature

按键②调节 36.0~39.0 摄氏度 Press button② to switch the alarm temperature between 36°C~39°C

按 4 下按键①：调音量 Press button① 4 times to set the volume

按键②调节 五档音量 可调 U0-U4 Press button② to switch the volume from U0 to U4

按 5 下按键①：调语言 Press button① 5 times to set the language

按键②切换 CH/EN/KO/PT/ES/JP/IT/DE/FR/RU/NL/AR/OFF 12 国语音 Press button② to switch language  
 CH/EN/KO/PT/ES/JP/IT/DE/FR/RU/NL/AR/OFF

CH: Chinese EN: English KO: Korean PT: Portuguese ES: Spanish JP: Japanese IT: Italian

DE: German FR: French RU: Russian NL: Netherlands AR: Arabia

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Tripod mounting: There is screw connection on the bottom which can be connected with tripod. (Fig.10)

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Wall-hanging: There are two hanging holes on the back side which can be fixed with nail. (Fig.11)

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(3) 桌面支架：用桌面支架放置桌面使用如（图 12） Desktop stand: Put the product on the flat surface with desktop support. (Fig.12)

(4) 支架挂壁：利用支架背部有两个挂孔可用挂钉固定如（图 13） Wall-hanging Desktop stand: There are two hanging holes on the back side of the desktop stand can be fixed with nail. (Fig.13)

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6. 电池安装：打开电池盖，拿出电池盒，放入 2 节 18650 锂电池，注意电池的方向（图 16、图 17、）  
Battery installation: Open the battery cover, put 2 \*18650 li-ion battery into the battery hole and insert as shown in pictures, then tighten up with screws provided. (Note: Do not insert battery backwards) (Fig. 16, 17)

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Warnings:

1. Do not pour hand sanitizer into the gap between Temperature Measuring Probe and Proximity Sensor.
2. Make sure the device is standing upright after filling with hand sanitizer.