



UK Version EmeTerm Explore Anti-nausea Wristband

Emeterm is a healthcare solution trading and manufacturer focusing on wearable and treatable electronic wrist device for anti-nausea and vomiting and extending to medical consumables. We consist of leaders in the field of medical research and product development who are devoted to provide effective and convenient therapy options to users

around the world now. We have over 10 years experience in medical business, approved with FDA, UKCA and ISO etc. UK Version EmeTerm Explore Anti-nausea Wristband exported to over 20 authorized distributors worldwide, covering markets in more than 30 countries or regions including the United States, Canada, the United Kingdom, Australia, and Japan. We are looking for more business partner and cooperation around the world .

UK Version EmeTerm Explore Anti-nausea Wristband

Emeterm is a healthcare solution trading and manufacturer focusing on wearable and treatable electronic wrist device for anti-nausea and vomiting and extending to medical consumables. We consist of leaders in the field of medical research and product development who are devoted to provide effective and convenient therapy options to users around the world now. We have over 10 years experience in medical business, approved with FDA, UKCA and ISO etc. UK Version EmeTerm Explore Anti-nausea Wristband exported to over 20 authorized distributors worldwide, covering markets in more than 30 countries or regions including the United States, Canada, the United Kingdom, Australia, and Japan. We are looking for more business partner and cooperation around the world .



UK Version EmeTermExplore Anti-nausea Wristband introduction

UK Version EmeTermExplore Anti-nausea Wristband is a safety and effective anti-nausea wristband that prevents and relieves nausea and vomiting induced by it can be used for various conditions of dizziness with zero side-effects. We provide a one year warranty for all products. It is covered with a waterproof technology and 30 hours battery time for outdoor activities. Electric Plates come out a non-invasive neuronal electrical stimulation go through brain to prevent brain transmitting nausea signal to stomach, then reduces the nauseous sensation for the users. Immediately effective for the treatment of nausea and vomiting caused by motion sickness (while in a car, a boat, or a plane), PONV (post operative nausea and vomiting) and morning sickness (during pregnancy) with zero side-effects. Our product is easy to install and charging. We have



www.watmedical.ca

helped a lot people suffering from motion sickness. They felt absolutely no nausea and enjoy life now.

Anti-emetic wristband that uses targeted neuromodulation to reduce nausea-induced vomiting. Hypoallergenic medical grade silicone with IP67 water and dust protection. Rapid onset with no medical side effects. Effective for motion sickness, morning sickness, drug-induced nausea, and more.

EmeTerm is a clinically-proven medical device and has been certified by the United States, Canada, Australia, Israel and the European Union. EmeTerm uses a targeted nerve regulation mechanism to release high-precision electrical pulse signals to the median nerve. It relieves nausea and vomiting caused by motion sickness, pregnancy, chemotherapy, and surgery. EmeTerm is free of side effects such as drowsiness, tremor and many others caused by medication. It can be used repeatedly, with IP67 waterproof grade, and a battery life of up to 30 hours of continuous usage



Product Specifications:

Brand	EmeTerm
color	Black, Blue
Principle	TENS Targeted Neuromodulation
Indications	Effective relief of all forms of nausea& vomiting including morning sickness, sea sickness, car sickness, VR sickness, air sickness, vertigo.
Age Range	The EmeTerm device is indicated for the treatment of nausea and vomiting in patients 6 years of age or older.
Certificate	FDA, CE, TGA, HC, ISO
Award	IF Design 2017
Notice	Indications vary due to regulations. Please check with customer service and local suppliers.

Success Rate:

US Version EmeTerm 1 Anti-nausea Wristband With an 85% success rate according to our clinical trials, the EmeTerm Anti-nausea Wristband is the FDA-cleared, all-natural wearable solution for nausea and vomiting.

Patented Conduction Enhancement Electrodes

Increased contact area

Better conduction

No gel required

Rechargeable Magnetic Charging

Equipped with a magnetic USB charging cable that can be quickly charged anytime and anywhere conveniently.

Only one charge is required for short trips.

HOW TO USE US Version EmeTerm 1 Anti-nausea Wristband?

STEP 1

Position: 2-3cm away from the transverse wrist line, with a width of about 2 fingers

STEP 2

Wear the device, Ensure skin is sufficiently in contact with electrodes

STEP 3

Turn on Home key to start the device, then adjust level until numb your palm.

Principle of US Version EmeTerm 1 Anti-nausea Wristband

Nausea and vomiting are caused by transmission of a nausea signal to the stomach via the human brain, which causes the stomach to contract irregularly, resulting in vomiting. By releasing a particular low-frequency pulse, EmeTerm can adjust the vagus nerve signals traveling to and from the stomach, and can prevent or postpone it from receiving nausea signals. This then slows down the irregular gastric contraction, and reduces the nauseous sensation for the users.

1. TENS consists of small unit, attached to the wrist via electrode pads, delivers signal through the median nerve at the location on the underside of the wrist.
2. These signals travel through the body's nervous system to the part of the brain which controls nausea, which causes the stomach to contract irregularly, resulting in vomiting.
3. By releasing a low-frequency pulse, it can adjust the vagus nerve signal traveling to and from the stomach, and can prevent or postpone the brain from transmitting nausea signals to the stomach, then reduces the nauseous sensation for the users.